BR | ES | EN | FR | IT Professional | Profesional | Professionnel | Professionale | Professionista

DENTSCARE LTDA - Av. Edgar Nelson Meister, 474 - Distrito Industrial - 89219-501 - Joinville - SC - Brasil / Brazil. CNPJ / Tax ID: 05.106.945/0001-06 INDÚSTRIA BRASILEIRA / BRAZILIAN INDUSTRY - Responsável Técnico / Technical Authority: Friedrich Georg Mittelstädt CRO: 13100147 - SC- ANVISA nº 80172310085 - Marca / Brand: FGM® / Fabricado no Brasil / Made in Brazil

ec rep NUNO FLORES Al. Bonifacio Lazaro Lozano, 3 - Piso 0 - C 2780-125 Oeiras / Portugal - Tel: (351) 21 4439292 - ec.rep@fgm.ind.br

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EN Instructions for use

Self-curing composite for provisional crowns and bridges.

For professional use only

Read carefully all the information in this document before using the product. Save it for later reference, at least, until total consumption of the product and/or until there is no more interaction between the product and its last patient.

Product description

PrimmaArt is a self-curing composite that is highly efficient for the production of temporary pieces for indirect dental procedures such as inlays, onlays, crowns, veneers and bridges. Another relevant recommendation is the possibility to provide previews to the rehabilitation by means of mock-ups, giving the professional the possibility to work with greater result predictability. The product consists of a base paste and a catalyzing paste stored in a cartridge and in a double-body syringe, from where they are automatically dispensed in the correct proportion (1:1), providing practical clinical procedures with reduced chances of errors or incorporated bubbles.

Commercial availability:

- Double-body syringe with 7g, 5 self-mixing tips. Available in shades A1, A2, A3, B1 and Bleach.
- Package with 5 disposable self-mixing tips for the double-body syringe.
- Package with 20 disposable self-mixing tips for the double-body syringe.
- Cartridge* with 75g, 10 self-mixing tips. Available in shades A1, A2, A3, B1 and Bleach. (*) Compatible with Universal Mixing Dispenser (1:1 and 2:1).
- Package with 5 disposable self-mixing tips for cartridge.
- Package with 20 disposable self-mixing tips for cartridge.

Basic composition

- Base paste: methacrylic monomers such as UDMA, Extended Urethane Dimethacrylate and TEGMA, Triethylene-glycol-Dimethacrylate, co-initiators, micro-particles of barium aluminosilicate glass, silicon dioxide particles, inorganic pigments and stabilizers.

- Catalyzing paste: methacrylic monomers, dibenzoyl peroxide, micro particles of aluminosilicate glass and stabilizers.

The mixture of the base and catalyzing paste in the 1:1 proportion presents 55 to 65% of charge in mass.

Product recommendations

- Production of temporary pieces: inlays, onlays, crowns, veneers and bridges.
- Production of long-term temporary pieces.
- Rebasing of prefabricated temporary crowns made of resin, polycarbonate or metal.
- Mock-up technique* (*) Mock-up is an intrabuccal restorative preview that allows the patient to pre-visualize the result of the work without any tooth wear being done.

Temporization:

	A	B	C	D
	40°	1° 30' - 2'	2-3'	5'
	Work time (product application)	Setting time at a temperature of 37°C/98°F	Setting time at room temperature (23°C/73°F)	Chemical curing time (complete setting)

Note: Differences in times may occur due to changes in room temperature. Those intervals may be reduced by higher temperatures and extended by lower temperatures.

Precaution and Contraindication

- The product should not be used in patients allergic to any of the substances contained in the formula.
- Use of protective gloves and a no-touch technique (self-mixing) is recommended.
- In case of contact with the skin, wash with soap and water.
- If accidental contact with eyes or prolonged contact with oral soft tissue occurs, flush with large amounts of water. If irritation persists, consult a physician.
- The setting time in the mouth (37°C/98°F) is approximately 2 minutes. The process must be monitored by observing the excess material in the oral cavity.
- The setting time of the unused material (in mixing tips) does not coincide with the hold time of the material in the mouth.
- Materials containing eugenol may influence the polymerization of this material. Therefore, they should be avoided.
- The self-mixing tip is for single use. Discard it after use.
- Make sure to place the cap in the double-body

polishing paste Diamond Excel (FGM).

Cementing:

- Clean and dry the prepared tooth, removing any remains of the isolating material.

- For cementing the temporary restoration, it is possible to use any temporary cement that does not contain eugenol. Note: Cements containing eugenol may prevent the polymerization of resinous cements during the permanent cementation.

Relining, Repairs and Complements

1) Temporary pieces not made with the product:
- Check the roughness and make retentions in the area to be repaired/relined with the help of drills.

- Apply an adhesive system (Ex.: Ambar, Ambar APS or Ambar Universal APS – FGM) according to the product instructions.

- Apply PrimmaArt all over the surface of the temporary element.

- Position the temporary piece on the tooth pressing the parts together for 2 minutes.

- After the chemical curing time of the product (approximately 5 minutes), prepare the temporary piece as usual.

2) Temporary pieces recently made with the product:
- It is not necessary to make retentions or to use the adhesive system.

- Apply PrimmaArt all over the surface of the temporary piece.

- Position the temporary piece on the tooth pressing both parts together for 2 minutes.

- After the chemical curing time of the product (approximately 5 minutes) prepare the temporary piece as usual.

Conservation and storage

- The packaging of the product must be closed immediately after use.

- Do not expose the product to high temperatures or intense light.

- Store the product at temperatures between 5°C and 25°C / 41°F and 77°F

- Expiration date is printed on the packaging.

Warnings

- All dental product should be used only by dentists, dental prosthetists, and authorized dental laboratories.

- Do not use the product after it is expired.

- When disposing the product, follow the legislation of your country.

- Keep out of the reach of children.

This material has been manufactured for dentistry use only and must be handled according to the instructions for use. The manufacturer is not responsible for damage caused by other uses or for incorrect handling. In addition, the user is obliged to prove, before use and under his responsibility if this product is compatible with the intended use, especially when such use is not indicated in these instructions for use. Data descriptions do not constitute any type of warranty and, therefore, have no binding.

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